



CD DISTALIZER U/L

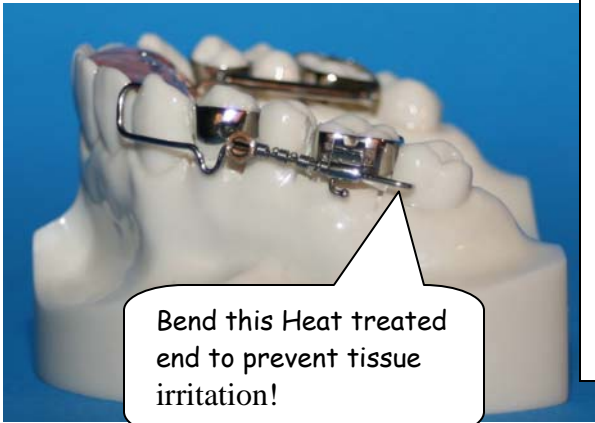

Bend this Heat treated end to prevent tissue irritation!

Dr. Chings CD Distalizer is a fixed orthodontic appliance used unilaterally or bilaterally to distalize molars on either the upper or lower arch.

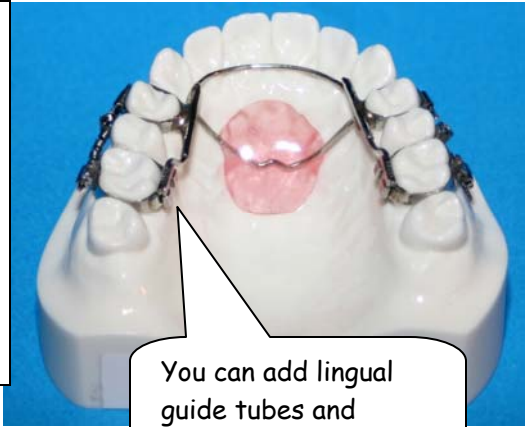

You can add lingual guide tubes and Wilson Attachments!

UPPER BI-LATERAL CD DISTALIZER

Why Doctors love Five Star's CD Distalizer:

We use a stronger wire (.036), which will give you less breakage and more bodily movement to the molars, with less crown tipping. We have added an optional lingual guide tube (see photo) to deter molar rotation during distalization. We can also add a midline screw in the anterior region, which gives the appliance the ability to expand the arch laterally. Five Star gives you the choice of soldered lingual buccal tubes or make yours with Wilson attachments for easy transition to a transpalatal arch.

Why Patients love Five Star's CD Distalizer:

Choose between vertical tubes on the bicuspids for easy segmental placement or have the guide wire soldered directly on the band for greater patient comfort. Five Star's anterior placement of the lingual metal tubing is tongue friendly eliminating sharp protrusive distal wire ends. Five Star bends the distal buccal .036 wire to greatly reduce cheek irritation!


How do you activate the CD Distalizer?

The CD Distalizer is easy to activate. At each patient visit, approximately every three to four weeks, the Gurin lock is loosened and pushed distally to compress the coil spring. When the spring is compressed, the lock is tightened to keep the coil spring activated. Repeat this procedure at each patient visit until the molars are in their desired position. When you have achieved the desired distalization, simply remove the spring and abut the Gurin lock to the Molar band to prevent relapse.

What do I need to get started?


Upper models from a heavy body alginate are fine. The models need to be free of bubbles, distortions, or voids. You can use PVS. The Doctor needs to remove all brackets and bands to ensure the teeth are clean prior to impression. Please check and wrap models individually before sending them.

Gain up to 1mm of molar Distalization a Month!

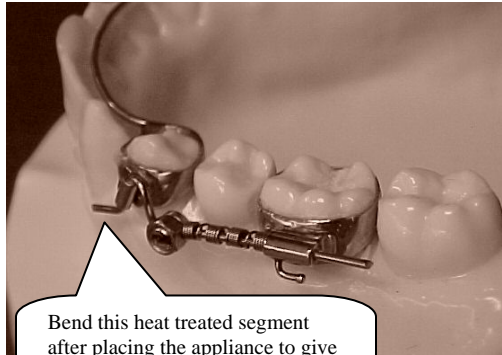

Bend this heat treated segment after placing the appliance to give further retention and patient comfort!

CLOSE UP OF THE LOWER CD DISTALIZER

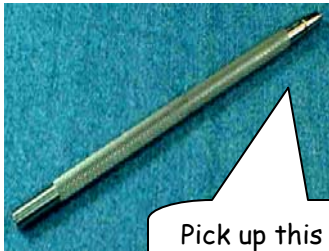

Pick up this wrench for easy appliance activation